

**PASSCHECK URINE CALIBRATOR  
MICROALBUMIN**

FOR USE IN SEMIAUTOMATIC  
INSTRUMENT

**Bialex®**

2°C 8°C

Used by:  
**2026.11.13**

Manufacture by Bialex Lab  
717 Ohio Ave  
Wichita Falls, TX 76301

Qty: 1 mL

REF: GQ-CMAL-1

LOT: 25CMAL0201

Made in U.S.A.

See the Value of  
Your Lot Here



#### **INTENDUSE**

Bialex® PassCheck Urine Calibrator Microalbumin is intend for use in semi automatic instruments and spectrophotometer with filter of 340nm.

#### **STORAGE AND STABILITY**

This product will be stable until the expiration date when stored unopened at 2 to 8°C

Once opened is stable for a week storage at 2 to 8°C and 1 day at room temperature/

#### **PROCEDURE**

Ready for Use.

#### **WARNING**

Handle this product as potentencially infecious source material with The Good Practice of Laboratory

#### **ASSIGNMENT OF VALUES**

The assignment values data charts was derived from compare between fully automatic procedure and semi-automatic procedures, using knew values of fresh urine concentration the variation in compare with fully automatic procedure shown no more than 1% up or down.

PassCheck Urine Calibrator Microalbumin, LOT 25CMAL0201

| Analytes     | Lot Assigned of Reagent | Value      |
|--------------|-------------------------|------------|
| Microalbumin | 24MAU0701               | 5.06 mg/dL |
| Microalbumin | 24MAU0901               | Pending    |
| Microalbumin | 25MAU0201               | 5.66 mg/dL |

See update new lot assigned in  
[www.bialexlab.com/insertos](http://www.bialexlab.com/insertos)

[www.bialexlab.com/apps](http://www.bialexlab.com/apps) (U.S. Only)