



INTEND USE

Bialex® PassCheck Urine Calibrator Microalbumin is intend for use in semi-automatic instruments and spectrophotometers with filter of 340nm.

STORAGE AND STABILITY

This product will be stable until the expiration date when stored unopened at 2 to 8°C

Once opened is stable for a week storage at 2 to 8°C and 1 day at room temperature.

PROCEDURE

Ready for Use.

WARNING

Handle this product as potentially infectious source material with The Good Practice of Laboratory

ASSIGNMENT OF VALUES

The assignment of values data charts was derived from comparison with fully automatic procedure in compare with semi-automatic procedures, using knew values of fresh urine concentration the variation in compare with fully automatic procedure shown no more than 1% up or down.

PassCheck Urine Calibrator Microalbumin, LOT 25CMAL701

Analytes	Lot Assigned of Reagent	Value (mg/dL)
Microalbumin	25MAU0601	4.36
Microalbumin	25MAU0701	5.70
Microalbumin	25MAU0801	5.40
Microalbumin	25MAU0901	5.90
Microalbumin	25MAU1101	4.30

See update new lot assigned in

<https://platinum.bialexlab.com/files>

www.bialexlab.com/apps (U.S. Only)