



717 Ohio Ave,
Wichita Falls, TX 763001

PassCheck Urine Control for Microalbuminuria, Creatinine and others analytes, FORM 195 Rev No: New

Use: IVD

Item: GQ-MAU1-1 & GQ-MAU2-1

Intended Use

PassCheck Urine Control was designed for use in QC in urine human basis, many of reagent's brand, due to the similar characteristics and values could be inside of parameters.

Preparation

Ready to use.

Keep 2-8°C

After opened it is stable for 7 days

Note:

We recommend using new controls for each QC assessment.

Precaution

All specimens using in the PassCheck Urine Control should be handled in accordance with good laboratory practices using appropriate precautions as described in the CDC/NIH Manual, "Biosafety in Microbiological and Biomedical Laboratories! 2nd edition, 1988 HHS Publication No (CDC)88- 8395

PassCheck Urine Control Lot 24PMAU10901 & 24PMAU20901 Exp.2026.07.01 (YYYY/MM/DD)

Analyte	Average	Level II	Units	Average	Level I
Microalbumin	9.40	7.52-11.28	mg/dL	4.90	3.92-5.88
Creatinine	152	128-176	mg/dL	65	57.1-72.90
Magnesium	9.05	8.26-9.83	mg/dL	4.39	3.26-5.52
Calcium	10.9	9.99-11.8	mg/dL	6.86	6.38-7.34
Chloride	189	173-204	mEq/L	93.4	81.7-104



