



717 Ohio Ave,
Wichita Falls, TX 763001

PassCheck Urine Control for Microalbuminuria, Creatinine and other analytes, FORM 195 Rev No: New USE:
IVD

Item: GQ-CMAU1-1 & GQ-CMAU2-1

Intended Use

PassCheck Urine Control is Specifically formulated for quality control (QC) in human urine Testing across a variety of reagent brands. Its design leverages comparable characteristics and established values inherent in urine parameters, ensuring consistent and reliable performance. This control Solution facilitates the accurate assessments of analytical methods, helping to maintain the integrity of testing processes and uphold diagnostic Standards.

Preparation

Ready to use.

Keep 2-8°C

After opening it is Stable for 24 hours

Precaution

All Specimens using in the PassCheck Urine Control should be handled in accordance with good laboratory practices using appropriate precautions as described in the CDC/NIH Manual, "Biosafety in Microbiological and Biomedical Laboratories 2nd edition, 1988 HHS Publication No (CDC)88-8395

PaSSCheck Urine Control 1: Lot 26CMAU0701, Control 2: Lot: 26CMAU0702 Exp.2028.02.10 (YYYY/MM/DD)

Analyte	Average	Level II	Units	Average	Level I
Microalbumin	10	8.00-12.00	mg/dL	4.00	3.20 - 4.80
Creatinine	150	129-171	mg/dL	63.5	51 - 76
Magnesium	-----	-----	mg/dL	4.10	3.0-5.2

