



717 Ohio Ave,
Wichita Falls, TX 763001

PassCheck Multicalibrator for Biochemistry, FORM 195 Rev No: New

Use: IVD

Item: GQ-CAL-5

Intended Use

PassCheck Multicalibrator was designed for use with the reagent of the same brand Bialex and is comparable with specific lot, please check before use the lot number. That is not universal Multicalibrator, however many of reagent's brand, due to the similar characteristics and values could be works fine.

Preparation

Using Class, A volumetric pipette add DI Water (pH: 7.0) 5mL, let for 30 minutes, time to time stir gently.

Keep 2-8°C

After opened it is stable for 7 days, except for bilirubin, glucose, Alkaline Phosphatase (24 hours). Reconstituted Frozen Stability 30 days at -20°C < 10%

Note:

We recommend using new calibrators for each calibration in your automatic instrument

Precaution

All specimens using in the PassCheck Multicalibrator should be handled in accordance with good laboratory practices using appropriate precautions as described in the CDC/NIH Manual, "Biosafety in Microbiological and Biomedical Laboratories! 2nd edition, 1988 HHS Publication No (CDC)88- 8395

DBA Bialex, certifies all individual human plasma donor units used to manufacture this lot were tested by FDA approved methods and found to ne nonreactive for HBsAg, HIV-1, HIV-2, and HCV

(*) Previously typography mistake = now is ok

Serum analytical	Value	Units
Total Bilirubin (Dyazo) (23BT1101 to 06) (24BT0401) (24BT0801)	1.08 1.06 0.86	mg/dL
Direct Bilirubin (Dyazo) (23BD1201) (25BD0501) (24BD0901 @546nm)	0.61 0.61 0.64	mg/dL
Calcium	10.8	mg/dL
Chloride (Ref: GQ-Cl-60)	81.23	mEq/L
Cholesterol (24CHO0501) Cholesterol (23CHO0701) Cholesterol (24CHO0801) Cholesterol (24CHO0802)	126.00 130.66 120.48 114.92	mg/dL
Creatinine (Lots: 24CRE0401/02/03) Creatinine (24CRE0901) Creatinine (24CRE0902) Creatinine (24CRE0903)	1.06 (*) 1.083 1.083 1.11	mg/dL
Glucose (Lot: 24GO0501) Glucose (Lot: 24GO0901) Glucose (Lot: 24GO0902)	112.00 84.30 94.90	mg/dL
Magnesium	2.92	mg/dL
Potassium	3.65	mEq/L
Total Protein	19.31	g/dL
Sodium (GQ-Na-100)	138	mEq/L
Triglycerides (24TT0201) Triglycerides (24TT0901) Triglycerides (24TT0902)	102.06 106.31 85.00	mg/dL
Uric Acid (Lot: 24UA0701)	3.52	mg/dL
Urea BUN (24BUN0901) Urea BUN (24BUN0902) Urea BUN (24BUN0903)	12.74 (*) 14.72 14.69	mgdL
Alkaline Phosphatase	82.40	IU/L
Amylase	239	U/L
AST Monoreagent (Biolis 24i) AST Monoreagent	98.72 76.92	U/L
ALT Monoreagent (AA Biolis 24i) ALT Monoreagent (24ALT0801) ALT Monoreagent (24ALT0901,02,03)	49.20 45.43 42.42	U/L
GGT (Lot: 24GGT0501) GGT (Lot: 24GGT0701) GGT (Lot:24GGT0901)	44.13 46.92 41.96	U/L
LDH (Lot: 24LDH0701) LDH (Lot: 24LDH0801)	244.44 243.40	U/L
Latic Acid	159	mg/dL
HDL DIRECT	SEE IN LIPPAS	
LDL DIRECT	SEE IN LIPPAS	

Albumin	8.90	gr/dL
CO2	PENDING	
Lipase Kinetic (Monoreagent) LOT 24LK0801 LOT 24LK0802	56	U/L